

# Initial set-up of your




## GoQuick Instruction Mat

This set-up procedure will **only need to be done once with each new pen**. After that, you only need to follow 3 easy steps for daily use (see overleaf).

Always wash and dry your hands before you start.

### 1. Attach needle, mix Genotropin®, and remove air

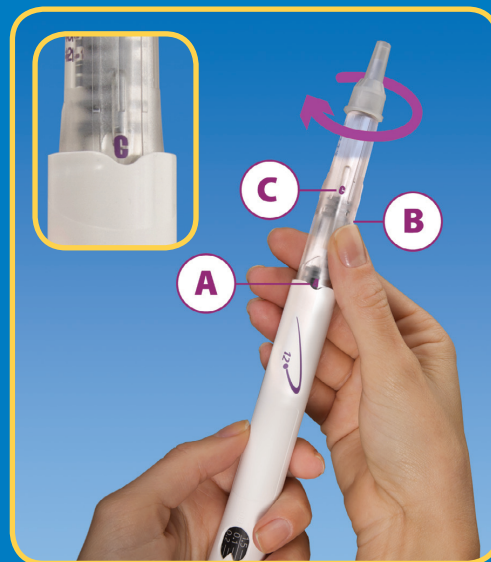

- Attach the needle.
- Hold the pen with **A** facing you. Firmly twist the cartridge holder clockwise until **B** clicks into notch.
- Gently tilt the pen from side to side. Do not shake it.
- Remove the outer needle cover. Leave inner needle cover on.
- Tap the cartridge holder gently.
- Firmly twist the cartridge holder clockwise until **C** clicks into the notch.

### 2. Attach needle guard (optional)



- Remove the **BLACK** cap from needle guard.
- Line up the **BLACK** logo with the pen logo and click into place.

### 3. Prime pen



- Remove the inner needle cover.
- Check that the lowest allowed dose is in the memory window.
- Turn the **GREY** dial in the direction of the arrows until it stops clicking.
- With the needle pointing up, push the **INJECTION BUTTON** to release the liquid.
- If the needle guard is attached, press the **BLACK** button on the back of the needle guard to release the needle shield.

### 4. Set dose




- Turn the **BLACK** ring until the dose lines up with **WHITE** pointer.
  - Be careful not to turn the **GREY** dial.
- NOTE: If you cannot turn the **BLACK** ring, press the **INJECTION BUTTON** until it stops clicking and try again.

### 5. Dial up dose





- Turn the **GREY** dial in the direction of the arrows (clockwise) until it stops clicking.
- The dose on the **BLACK** rod should be the same as the dose in memory window.

### 6. Give injection

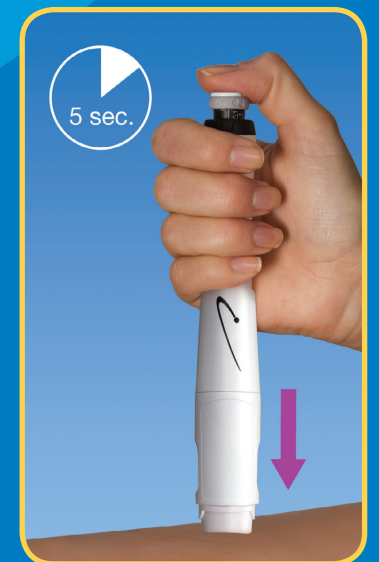

- Prepare the injection site.
- Push the pen down on injection site to insert needle into skin.
- Push the **INJECTION BUTTON** down until it stops clicking. Wait 5 seconds before you pull the needle out.
- Place the outer needle cover into the end of the needle shield
- Use the needle cover to push in the needle shield until it locks into place
- Use the needle cover to unscrew the needle and put it in a proper container for used needles
- Leave the needle guard on the pen
- Put the black cap on the needle guard. Store your pen in the refrigerator

## Your **GoQuick**<sup>™</sup> Genotropin (somatropin, rbe) pre-filled pen

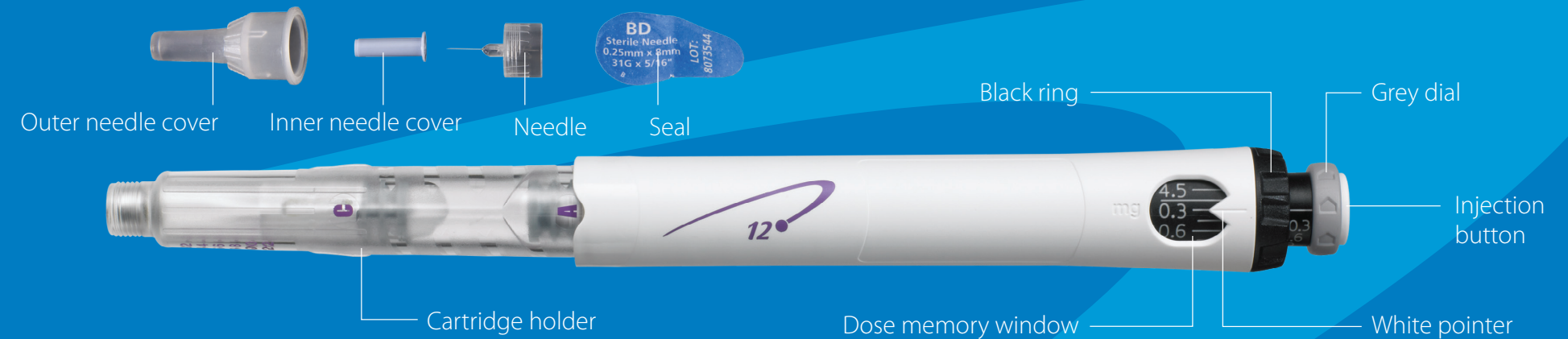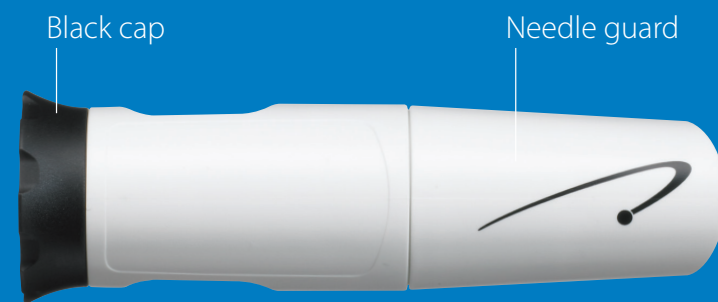


## 3 easy everyday steps

### 1. Attach the needle



### 2. Dial up the dose

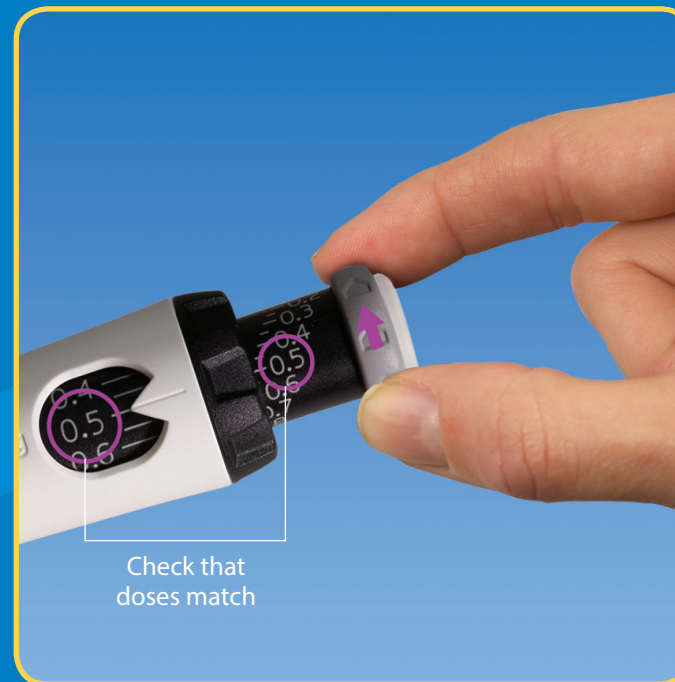

Make sure the dose you have dialled up on the **BLACK** rod is the same as the dose you set in the memory window.

### 3. Give the injection



Remove the needle, replace the cap (black cap with needle-guard, white cap without) and store in the refrigerator.

# Non-Promotional Certificate UK

**Pfizer Sponsor**      James Steed

**Name**                      UK\_GIP\_Endocrine Care\_Genotropin GoQuick training mat (with needle guard)

**Document Number** PP-GIP-GBR-1082

## NON-PROMOTIONAL ITEM CERTIFIED FOR USE

The below signatories have examined the electronic final form of this material and in their belief it complies with the Code.

Other Final Country Signatory	Name: Pankaj Maini Signature Date: 08-Dec-2016 12:43:05 GMT+0000 Reason: Non-Promotional item certified for use
Other Final Country Signatory	Name: STEVEN YULE Signature Date: 30-Oct-2018 23:50:05 GMT+0000 Reason: Non-Promotional item certified for use
Medical Affairs	Name: Carolyn Whitlock Signature Date: 02-Nov-2020 09:07:04 GMT+0000 Reason: Non-Promotional item certified for use